

# POWER INFUSER<sup>®</sup> Quick Start Guide

**1** Remove Contents

**2** Pump Controls

**3** Install AAA Batteries

**4** Make Connections To Cartridge

**5** Install Cartridge

**6** Turn On Prime/Eliminate Air

Note: Tilt unit, tap cartridge to remove air.

**7** Infuse IV Fluids

Attach to patient  
18 ga optimal

BOLUS 250cc in 2.5 min

Set flow rate

0.2L/h, 1L/h, 2L/h, 4L/h, 6L/h

BLOOD, DRUGS

**8** Exchange IV Bag

Note: Clamp IV set prior to exchange.

**9** End Infusion

**10** Free Flow if Desired

Note: Cartridge is single-use disposable.